

Perineal Post Cover

*Quick, easy setup and
increased infection control.*



Features:

- Integrated drawstring seals cover around perineal post sidebar
- Integrated tape strips (qty 8) securely hold the cover in place
- Clear polyethylene material - not made with natural rubber latex
- Non-sterile, single-use
- 12" wide x 28" long




Individual Cover



Box of 10


Instructions:

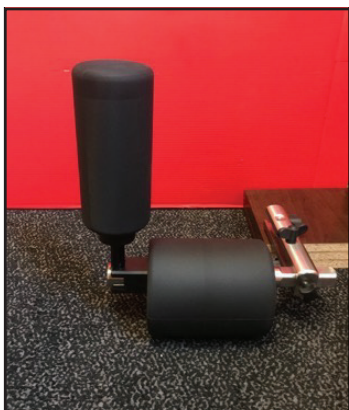


1. Set femoral support to desired setting.



2. Put cover on with labels facing away from the OR table.

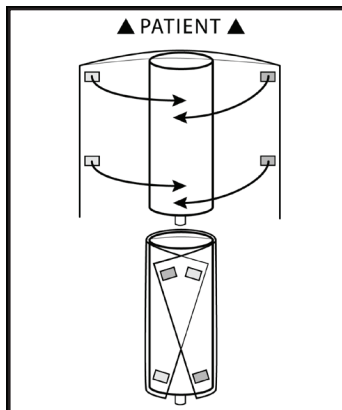

3. Secure top four tape strips around perineal post.

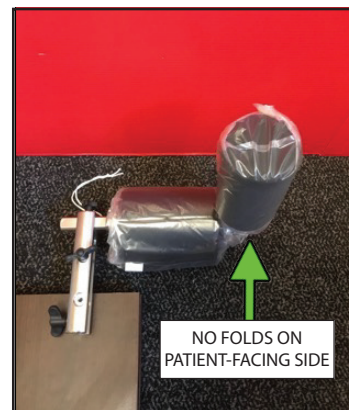

4. Secure drawstring. Use lower tape strips to further secure cover.

IOT (Innovative Orthopedic Technologies) develops specialty solutions for the orthopedic market, with a focus on innovative patient positioning devices that fit seamlessly into the operating environment. We aim to streamline procedures, support repeatability, and literally free the hands of the surgical team.

Unleash your surgical genius.



United States:
Innovative Orthopedic Technologies, LLC
1115 Hwy 124 · Winnie, TX 77665
t: +1 409-658-1017 · e: info@iotiot.com · w: www.iotiot.com

International:
Innovative Orthopedic Technologies IOT AG
Schochenmühlestrasse 4 · CH-6340 Baar · Switzerland
t: +41 41 749 4099 · e: info@iotiot.com · w: www.iotiot.com

20001 Rev A 042621 © 2021 Innovative Orthopedic Technologies, LLC